Only handwash up to 30 times at 30 degrees washable, wash after 7 days at the latest.

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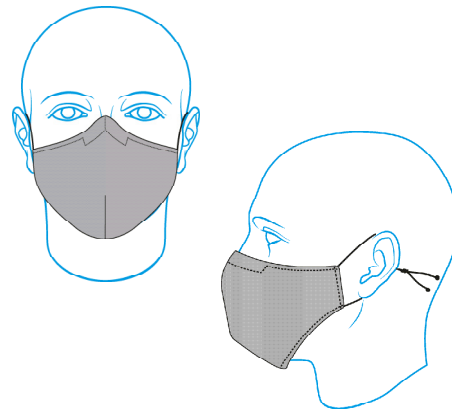
**Instructions for the proper fitting of the ZWISSMASK**

1. Wash and disinfect hands thoroughly (20-30 seconds with soap) before and after putting on the mask.
2. Unfold the zwissMASK.
3. Take the two elastic straps on each side and knot them together so the zwissMASK fits tightly when you put it on in the next step.
4. Place the zwissMASK on your face and put the ear loops around your ears.
5. Adjust the nose clip tightly around your nose and press it together. The nose clip should only be bent, but not kinked, otherwise it may break.
6. Check the fit on the chin, mouth and nose and adjust the position of the zwissMASK when it is necessary.

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\*The zwissCLEAN® treated upper covering the filter media on the front side has a very high antimicrobial activity (>99.9%) against a wide range of microorganisms, including Escherichia coli as a test organism. The antimicrobial activity was tested according to AATCC 100, DIN EN ISO 20743 and DIN EN ISO 18184. The fibres contain a chemical formulation that has an antimicrobial effect through a physical reaction. In the fibre polymer this counteracts the colonisation, replication and odour development of microorganisms on the treated products.

**3 SPEZIELLE SCHICHTEN**

**1 Innenstoff** 100 % BAUMWOLLE

**2 Mittelstoff** 100 % TPU-VLIES

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**1**  
**100% COTTON**  
3 SPECIAL LAYERS

**2**  
**FILTER FLEECE**

**3**  
**ANTIMICROBIAL TEXTILE**